- en • Ostomy Belt
- cs • Stomický pásek
- da • Stomibælte
- de • Stomagürtel
- fi • Avannevyö
- fr • Ceinture de stomie

- hu • Sztómazásákrögítő öv
- it • Cintura per stomia
- ja • ストーマ用ベルト
- lt • Stomas diržas
- nl • Stomagordel
- no • Stomibelte

- pl • Pasek stomijny
- ro • Curea pentru stomă
- sk • Stomický pás
- sl • Pas za stomo
- sv • Stomibälte
- zh-TW • 造口繫帶

en • INSTRUCTIONS FOR USE

Contents

- Ostomy belt

Product Description

Adjustable belt used with ostomy pouches that have belt tabs. Product is made of elastic and polymer/copolymer plastics.

Indications and Intended Use


This product is an accessory to an ostomy pouching system intended to aid adhesion, provide an extra sense of security, or aid the degree of convexity when used with a convex skin barrier.

Precautions:

Single patient – multiple use. Sharing this product with multiple users creates a risk of infection. Replace when the elastic has lost its stretch.

Do not overtighten the belt. Allow two fingers width between the belt and the skin.

To apply

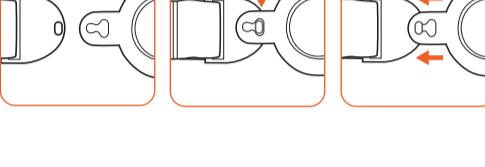Notes:

Product fit is affected by many factors such as stoma construction, stoma output, skin condition, and abdominal contours. Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.dansac.com/authority or contact local distributor.

To dispose (Please follow all local disposal regulations)

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