# NovaLife

## nl • 2-delig open opvangzakje

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#### Inhoud

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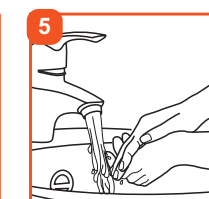
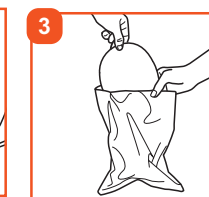
+45 48 46 50 00  
[www.dansac.com](http://www.dansac.com)

dansac

For reference only. Unauthorized use, disclosure or copying is prohibited.

## NovaLife

**en** • **2-Piece Open Pouch**

**INSTRUCTIONS FOR USE**

**Contents**

- Drainable pouch with attached coupling ring
  - Ostomy belt tabs (belt not included)
  - Integrated closure

**Product Description**

Part of a two-piece ostomy pouching system. Designed to be used with a corresponding skin barrier with the same coupling ring size. The pouch is designed to be emptied. The pouch is most suitable for colostomies or ileostomies. Product is made of polymer/copolymer plastics, and nonwoven fabric.

**Indications and Intended Use**

This product is part of a two-piece ostomy pouching system intended to collect output from a stoma (e.g., colostomy, ileostomy) or skin opening (e.g., wound, fistula).

**WARNINGS**

Reuse by reprocessing, cleaning, disinfection, and sterilisation may compromise the product. This may result in odour or leakage.

**Notes:**

Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority.

For more information, see www.dansac.com/authority or contact local distributor.

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