## nl • Nachtopvangzak

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#### Inhoud

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**MD** Medisinsk enhet  
(dersom aktuelt)

**For reference only. Unauthorized use, disclosure or copying is prohibited.**

**en • Night Drainage Bag**

**INSTRUCTIONS FOR USE**

**Contents**

- Night Drainage Bag with tubing and attached connector.

**Product Description**

Night drainage bag that connects directly to a Dansac urostomy pouch without the need for additional connectors. Features punch holes for attachment to a nighstand or bed hanger (not included). Designed to be emptied. Most suitable for use for urostomies.

Product is made of polymer/copolymer plastics, and nonwoven fabric.

**Indications and Intended Use**

This product is an accessory to an ostomy pouching system intended to collect urine from Dansac urostomy pouches.

**WARNINGS**

Disinfection and sterilisation may compromise the product. This may result in odour or leakage.

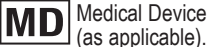
**Precaution**

Single patient – multiple use. Empty then rinse bag and tubing with clean water, and hang to dry. Wipe outside of bag and tubing with damp cloth as needed. Discard when visibly damaged or stained. Sharing this product with multiple users creates risk of infection.

**Notes:**

Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority.

For more information, see www.dansac.com/authority or contact local distributor.

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