INSTRUCTIONS FOR USE
en • Connect the stoma cone tube to the tube from the water container by pushing them together. Ensure the flow control is closed. Fill the water container with the volume of lukewarm water as directed by your stoma care nurse or healthcare professional (typically 500-1000 ml).

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Hang the water container ensuring it is positioned at shoulder height or higher when seated. Open the flow control to allow water to pass through the cone, removing air from tubing. Close flow control.

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en • You may wish to move around as you wait for stoma to function. To do so, close both the top and bottom of the irrigation sleeve with the pegs provided.

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Normally, your colon will empty in 30-45 minutes, but timing may depend on frequency of irrigation.

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Attaching the irrigation sleeve:
A 1-piece system - Remove the protective film from the back of the adhesive skin barrier on the irrigation sleeve and attach the skin barrier to fit around the stoma.

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Roll out the irrigation sleeve placing the lower end into the toilet to empty contents.

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B 2-piece system - Place the silicone ring in the ring holder of the irrigation sleeve. Attach the ostomy belt to one side of the ring holder and position the ring around the stoma. Wrap the ostomy belt around the body and attach it to other side of ring. Adjust ostomy belt to make a watertight seal against the skin.

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Once discharge from stoma is complete, remove the irrigation sleeve and clean around your stoma and peristomal skin with lukewarm water and soft tissue.

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Sit on toilet or chair next to toilet, placing the end of the irrigator sleeve into the toilet bowl. Gently insert stoma cone into the stoma through the top opening of the irrigation sleeve. Insertion of the stoma cone may be eased using water or a stoma lubricant.

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Ensure that the skin is dry and then attach a new appliance. Until colostomy irrigation is well regulated, you should apply a pouch in case the stoma functions between irrigations. Once irrigation regime is established, a stoma mini cap may be worn.

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Open clamp, regulating the flow of water to the desired speed. Flow slowly into the stoma over 5-10 minutes. Close clamp when sufficient water has entered your colon and remove stoma cone from the stoma.

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Disconnect and clean all irrigation parts with water. You may use the cleaning brush if included with your set. **To dispose** (Please follow all local disposal regulations). **DO NOT FLUSH DOWN THE TOILET**

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Dansac A/S
 Lille Kongevej 304
 DK-3480 Fredensborg

+45 48 46 50 00
 www.dansac.com

en	Stoma Irrigation Set		
REF	Contents		
	Irrigation Set content may vary per market.		
Irrigation Set Products			
09420-0000	Dansac toilet bag		
95200-0000	Water container		
95225-0000	Tubing with connector and regulation clamp		
95205-0000	Stoma cone with tubing		
95220-0000	Cleaning brush		
09547-0000	Silicone ring		
950-20	Irrigation sleeve with ring holder and silicone ring		
R4012-2	Pegs		
950-35	Irrigation sleeve with adhesive skin barrier		
Additional Products			
09000-0000	Ostomy belt		
70005-0001	Dansac Skin Lotion		
71000-0000	Dansac Skin Lotion Tissues		
903-10	Dansac NovaLife™ 1-Piece closed pouch		
829-20	Dansac Nova™ Mini Cap		

www.dansac.com/symbolglossary

Product Description

Products to support colostomy irrigation. The irrigation sleeve is designed to be adhered to the skin around the stoma or held in place with a ring and ostomy belt to help direct stoma output into toilet. The water container with stoma cone and tubing are used to flow water into the stoma. The pegs may be used to help close the irrigation sleeve. The cleaning brush is used when rinsing the cone and tubing. Devices are made of polymer/copolymer plastics, silicone, and may contain metal.

Indications and Intended Use

These products are intended to be used for colostomy irrigation. Irrigation is used to help regulate bowel emptying when recommended by a stoma care nurse or healthcare practitioner.

WARNINGS

Use of extremely hot water may cause injury. Complications may also include non-return of fluid, vasovagal response, and abdominal cramps.

	Reuse of a single use product may create potential risk to the user such as poor adhesion. Reprocessing, cleaning, disinfection, and sterilisation may compromise the product. This may result in odour or leakage.
	MR Unsafe. This device has been tested for safety in the Magnetic Resonance (MR) environment. This device is labeled MR Unsafe and should not be used in an MR environment.

Precautions

The irrigation kit is not intended for use with a urostomy or ileostomy. Injury and/or infection may result if products are used with a urostomy or ileostomy.

For products indicated as:		Single patient - multiple use. Rinse with clean water, and wipe dry as needed. Discard when visibly damaged or stained. Sharing the product with multiple users creates risk of infection.
----------------------------	--	--

Notes

Irrigation may not be recommended if you have any of the following conditions: current chemotherapy, radiotherapy, stoma stenosis , large parastomal hernia , stoma prolapse, Crohn's disease, or diarrhoea. Contact a stoma care nurse or healthcare practitioner to decide if irrigation is right for you, and for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.dansac.com/authority or contact local distributor.

Please turn over page to see instructions for use.

			
			
			
			
			
			
	Medical Device (as applicable).		Medical Device (as applicable).
Not made with natural rubber latex.			

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