
INSTRUCTIONS/使用说明/GEBRUIKSAANLEITUNG/BRUGSANVISNING/INSTRUCCIONES / KÄYTTÖOHJEET  
DÉS DE L'EMPLOI/ÖÄHTHEI/UTÁSI TÁSKÓK/PETÚNUUK/תְּרִינָה/ISTRUZIONI/使用方法  
언내/GERBUKSAANLIZING/BRUKSAANVISNING/INSTRUÇÕES/العِيَادَةُ/TALIMATLARI  
说明/HUANG DÂN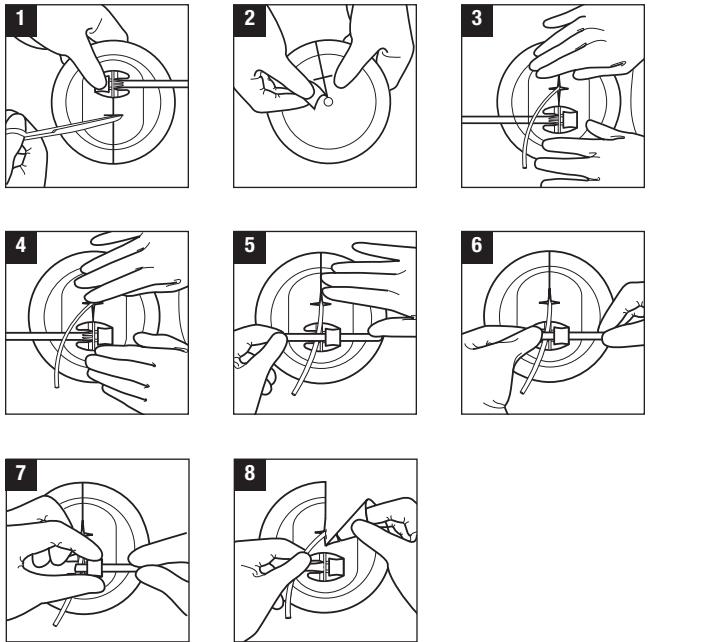

CE 0050

Hollister®

**US GB** Horizontal Drain/Tube Attachment Device

Fits 5-40 French

**WARNING:** To help ensure proper adhesion and to help reduce the potential for infection, do not reuse.

The Hollister horizontal drain/tube attachment device, a sterile, flexible support secured to a synthetic skin barrier with adhesive backing, is designed to stabilize (stabilise) and secure a variety of catheters, surgical drains, and tubes.

The horizontal drain/tube attachment device provides protection for the patient's skin in three ways:

- It eliminates the need for suturing certain tubes to the skin.
- It protects the skin surrounding the tube site.
- It minimizes tube movement which can lead to discomfort and enlargement of the tube entry site.

**Preparing the Skin**

Before applying the attachment device, wash the patient's skin with Hollister skin cleanser or any mild soap and water. Rinse thoroughly and dry completely. Do not use any lotions or emollients as the residue will interfere with the adhesion of the barrier.

**Applying the Attachment Device**

1. Using sterile technique (where applicable), cut a cross slit (a) in barrier perpendicular to the starter slit (b). The larger the diameter of the drain/tube, the farther away from the drain/tube holder the cross slit should be made. If needed, trim the barrier for a closer fit around the tube.
2. Peel the release paper from the skin barrier on the underside of the attachment device. Do not remove the release paper from the tape border.

3. Feed the drain/tube through the starter slit to the cross slit. Position the device so that the drain/tube holder is below the tube exit site. Press the barrier to the skin firmly, but gently, and smooth it to remove all wrinkles to ensure complete adhesion. Remove the release paper from the tape border and press tape to skin.

4. Place the drain/tube on the top of the serrated strap, making sure the drain/tube is not knotted.

5. Bring the strap over the tube and push it into the strap insertion hole in the drain/tube holder.

6. Pull the strap while pushing it from the other side until it fits snugly around the drain/tube. Be careful not to dislodge the attachment device during this step. The strap should squeeze the drain/tube slightly for maximum security.

**CAUTION:** The Hollister horizontal tube attachment device was designed as an alternative to tape. It does not eliminate any of the risks associated with the use of catheters, surgical drains, and tubes. For proper use of catheters, surgical drains, and tubes, be sure to consult the percutaneous tube product instructions and follow current standard clinical procedure for the placement, monitoring, and removal of percutaneous tubes.

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7. Release the drain/tube from the tube holder by pushing the serrated strap backwards while squeezing the sides of the latch. Pull the strap all the way back through the latch.

8. Starting at the cross-slit opening, gently peel the barrier and tape from around the drain/tube and from the patient's skin. Hold the tube with one hand and remove the attachment device.

**CAUTION:** Do not pull the serrated strap too tightly as the pressure may cause occlusion in or damage to the drain/tube.

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**2.** From the adjacent assembly bottom surface skin protect film tear off the insulation paper. Cut the tape skin side.

**3.** Insert the tube into the tube holder until the tube is fully inserted. Turn the tube holder clockwise to lock the tube.

**4.** Turn the tube holder counter-clockwise to unlock the tube.

**5.** Insert the tube into the tube holder until the tube is fully inserted. Turn the tube holder clockwise to lock the tube.

**6.** Turn the tube holder counter-clockwise to unlock the tube.

**7.** Insert the tube into the tube holder until the tube is fully inserted. Turn the tube holder clockwise to lock the tube.

**8.** Turn the tube holder counter-clockwise to unlock the tube.

**9.** Insert the tube into the tube holder until the tube is fully inserted. Turn the tube holder clockwise to lock the tube.

**10.** Turn the tube holder counter-clockwise to unlock the tube.

**11.** Insert the tube into the tube holder until the tube is fully inserted. Turn the tube holder clockwise to lock the tube.

**12.** Turn the tube holder counter-clockwise to unlock the tube.

**13.** Insert the tube into the tube holder until the tube is fully inserted. Turn the tube holder clockwise to lock the tube.

**14.** Turn the tube holder counter-clockwise to unlock the tube.

**15.** Insert the tube into the tube holder until the tube is fully inserted. Turn the tube holder clockwise to lock the tube.

**16.** Turn the tube holder counter-clockwise to unlock the tube.

**17.** Insert the tube into the tube holder until the tube is fully inserted. Turn the tube holder clockwise to lock the tube.

**18.** Turn the tube holder counter-clockwise to unlock the tube.

**19.** Insert the tube into the tube holder until the tube is fully inserted. Turn the tube holder clockwise to lock the tube.

**20.** Turn the tube holder counter-clockwise to unlock the tube.

**21.** Insert the tube into the tube holder until the tube is fully inserted. Turn the tube holder clockwise to lock the tube.

**22.** Turn the tube holder counter-clockwise to unlock the tube.

**23.** Insert the tube into the tube holder until the tube is fully inserted. Turn the tube holder clockwise to lock the tube.

**24.** Turn the tube holder counter-clockwise to unlock the tube.

**25.** Insert the tube into the tube holder until the tube is fully inserted. Turn the tube holder clockwise to lock the tube.

**26.** Turn the tube holder counter-clockwise to unlock the tube.

**27.** Insert the tube into the tube holder until the tube is fully inserted. Turn the tube holder clockwise to lock the tube.

**28.** Turn the tube holder counter-clockwise to unlock the tube.

**29.** Insert