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Instructions for use

Product description
Hydrocolí is a sterile self-adhesive wound dressing consisting of an adhesive hydrocolloid layer covered with a polyurethane film. The wound site is covered with a thin layer imposed of siliconated paper.

The primary packaging consists of a paper pouch made of film and coated paper. Hydrocolí is a non-medical device. It does not contain any medicinal substances, tissue or cells of human or animal origin or nanomaterials. It is sterilized using irradiation.

Composition
Hydrocolí consists of a gas permeable polyurethane cover. The hydrocolloid compound itself mainly contains sodium carbamylcellulose, a synthetic rubber matrix based on polyisoprene-polyisoprene copolymers and polyisobutylene.

Properties and mode of action

The product absorbs wound exudate. After the exudate has been absorbed, the hydrocolloid layer transforms into a gel which provides a moist wound healing environment. The ingress of germs is prevented by the polyurethane film. The film is water-repellent and allows moisture vapor and gas to pass through. Since the gel is produced during the application, the dressing does not adhere to the wound and the healing process is not impaired by the dressing change. Newly formed tissue is not damaged.

Due to the thin flexible material, Hydrocolí media easily to body contours. The surface of Hydrocolid is dirt-repellent and water-repellent.

Intended purpose

Hydrocolí is a sterile, single-use wound dressing for the treatment of medium to lightly exuding wounds such as chronic wounds (leg ulcers, decubitus, etc.) or burns up to second degree.
Use by healthcare professionals.
Accumulated use of the product can exceed 30 days.

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Hydrocolid est destiné au traitement des plaies de personnes de santé.
Mode of application
Select the correct size and shape of Hydrocolid. The dressing should exceed the wound by at least one centimeter to ensure that it stays firmly in place. If needed, cut the dressing to the size of the wound with sterile scissors.

Hydrocolid /Hydrocolid Think: remove the protective product cover, place Hydrocolid on the wound, adjust the edges to fit and press on.

Hydrocolid Sacral: remove the protective cover and press the dressing in the anal cleft. The top of the dressing can point upwards or downwards depending on the position and depth of the ulcer. Press the edges on firmly.

Hydrocolid Concave for the heel and elbow: remove the largest part of the protective product cover, place the middle part of the Hydrocolid dressing on the wound, remove the rest of the protective product cover, then overlap the wings by 10%. Due to the absorption of exudate a whitish gel is formed that is visible on the top as a discoloration and a blister. Hydrocolid should be changed if the blister has almost reached the edge of the dressing.
How Hydrocolid should be changed
The dressing should be changed if necessary. To change the dressing, lift Hydrocolid carefully from the dressing. It may be changed immediately.
How Hydrocolid should be changed
The dressing should be changed if necessary. To change the dressing, lift Hydrocolid carefully from the dressing. It may be changed immediately.

The wear time of the dressing depends on the condition of the wound and the level of exudate but can remain on the wound for up to 7 days.

Contraindications
Hydrocolí is contraindicated in any patients who may be allergic to any of its ingredients. Hydrocolid must not be used on wounds extending into muscle, tendon or bone. It must not be applied to clinically infected wounds or third-degree burns.

Product disposal
In order to minimize the risk of potential infection hazards, or environmental pollution, disposable components of the medical device should follow disposal procedures according to applicable and local laws, rules, regulations and infection prevention standards.

Special precautions
Hydrocolí is contraindicated in any patients who may be allergic to any of its ingredients. Hydrocolid must not be used on wounds extending into muscle, tendon or on os. It do not be applied to clinically infected wounds or third-degree burns.

Precautions
Use a single-use medical device is dangerous. Reusing devices, in order to re-use them, may seriously damage their integrity and their performance. Information available on request. Before opening the product packaging, inspect it carefully for signs of damage. Do not use the product if there are visible signs of damage, such as cracks, channels, pinholes, or nonuniform, torn or incomplete seals.

Hydrocolloid

Incident reporting
For a patient/user/third party in the European Union and in countries with identical regulatory regime (Regulation (EU) 2017/745 on Medical Devices) if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority.

Date of revision of the text: 2023-07-10
AU – PAUL HARTMANN Pty. Ltd. - Macquarie Park, NSW 2113
CA – PAUL HARTMANN (Canada), Inc. - 3255 Wiener Newford CR – WF HARTMANN GmbH - 82172 Weilhausen

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