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Instructions for use

Please read the instructions before use. The physician and medical staff are responsible for applying the casts and for treating patients in the required manner.

Product description
Saftex plus plaster of Paris bandages and slabs are designed for splints, as well as for external immobilization using circular plaster of Paris bandaging techniques.

Composition
Saftex plus plaster of Paris bandage consists of natural gypsum (CaSO4 ½ H2O) and a carrier material made from a specially woven cotton cloth.

Intended purpose

Saftex plus are single-use, non-sterile plaster of Paris bandages and slabs for short- and long-term treatment of external immobilization on humans using circular plaster of Paris bandaging techniques and for splints, used by healthcare professionals in a clinical environment applied in combination with products that are used as underlays.

Mode of application

Saftex plus is activated after having been immersed in water for about 3 seconds at approximately 20 °C (it is possible to cut the product before immersing it in water). It should then be wrung out thoroughly and moulded immediately. The moulding time is approximately 2–4 minutes. Saftex plus plaster casts and splinting bandages can support light loads after 30 minutes. After 12–24 hours, the product is completely hardened, depending on the number of applied layers.

Protect the entire part of the body being treated with suitable padding (synthetic wadding, tube, foam).

Special precautions
While setting Saftex plus, heat is released which is tolerated by the patient if the instructions for use are followed.

There is danger of burns due to excessive heat release under the following circumstances:

- water temperature higher than 27 °C during immersion
- bandage applied too dry
- multi-layered cast

obstruction of the circulation during setting by covering with a towel and other material

Please pay attention to these situations! Avoid covering completely!

Only for use on intact skin.

For a patient/user/third party in the European Union and in countries with identical regulatory regime (Regulation 2017/745/UE on the disposable medical devices), if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority.

Reusing a single-use medical device is dangerous. Reprocessing devices, in order to reuse them, may seriously damage their integrity and their performance. More information about this is available on request.

Patients should be warned not to be in contact with excessive amounts of moisture through immersion, for example while swimming or bathing!

Removing the casting bandages

Once the casting bandage has been removed, it can be removed with a cast saw and scissors. When using the cast saw, extra care must be taken not to injure the patient.

Product disposal

To minimize the risk of potential infection hazards, or environmental pollution, disposable components of Saftex plus should follow disposal procedures according to applicable and local laws, rules, regulations and infection prevention standards.

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Special precautions

While setting Saftex plus, heat is released which is tolerated by the patient if the instructions for use are followed.

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Removing the casting bandages
Once the casting bandage has been removed, it can be removed with a cast saw and scissors. When using the cast saw, extra care must be taken not to injure the patient.

Product disposal
To minimize the risk of potential infection hazards, or environmental pollution, disposable components of Saftex plus should follow disposal procedures according to applicable and local laws, rules, regulations and infection prevention standards.

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