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### Produktbeschreibung

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## EN Instructions for use

### Product description

Sorbalgon Classic/Sorbalgon T Classic alginate is made of natural fibres derived from seaweed. It is designed to be a highly absorbent and strong saugability. The alginate absorbs wound exudates and forms a gel-like covering over the wound, maintaining a moist wound healing environment which accelerates the healing process and helps to stop bleeding. The gel-like surface also keeps the dressing from adhering to the wound, so it is painless to remove.

### Composition

- calcium alginate
- sodium alginate

### Intended purpose and indications

Sorbalgon Classic/Sorbalgon T Classic is indicated for the management of moderate to heavily exuding partial to full thickness wounds, including:

- surface trauma
- cavity wounds
- pressure ulcers
- diabetic foot ulcers
- venous/arterial leg ulcers
- donor sites
- post-operative wounds

Sorbalgon Classic/Sorbalgon T Classic is intended to be used by lay persons and health care professionals.

### Mode of application

1. Clean the wound with saline solution and dry the surrounding skin of the wound gently.
2. Choose the right size of the dressing according to the dressing to the dressing to the wound bed correctly ab.
3. Choose the right size of the dressing according to the dressing to the dressing to the wound bed correctly ab.
4. Choose the right size of the dressing according to the dressing to the dressing to the wound bed correctly ab.
5. Choose the right size of the dressing according to the dressing to the dressing to the wound bed correctly ab.

### Removal:

1. Gently remove the secondary dressing.
2. If the wound appears dry, saturate the dressing with saline solution prior to removal.
3. Gently remove the dressing from the wound bed and discard.

### Contraindications

- Sorbalgon Classic/Sorbalgon T Classic is contraindicated on any patients who may be allergic to any of its ingredients.
- Do not use this product on dry wounds.
- The dressing is not indicated for surgical implantation.

### Precautions

- If redness or allergic symptoms appear, please stop using the product.
- Sorbalgon Classic/Sorbalgon T Classic is not recommended to control heavy bleeding wounds.
- Infected wounds should only be treated under the supervision of a healthcare professional.

### Product disposal

In order to minimize the risk of potential infection hazards or environmental pollution, disposable components of Sorbalgon Classic/Sorbalgon T Classic should follow disposal procedures according to applicable and local laws, rules, regulations and infection prevention standards.

### Special precautions

In the absence of available data supporting the use of this dressing on sensitive population groups such as infants, children, pregnant or nursing women, and in the absence of data to the contrary, on these population groups, this dressing should be used with caution following a clinician's recommendation.

### Sorbalgon Classic/Sorbalgon T Classic is meant for single use only.

Resealing, Reprocessing devices, in order to reuse them, may seriously damage their integrity and their performance.

### Before opening the product packaging, inspect it carefully for signs of damage.

Do not use the product if there are visible signs of damage, such as cracks, channels, pinholes, or nonuniform, torn or incomplete seals.

### Incident reporting

For a patient/user/third party in the European Union and in countries with identical regulatory regime (Regulation (EU) 2017/745 on Medical Devices); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority.

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